



# AESCULAP® DS Appendectomy-Clip

Innovative, reliable, convenient,  
cost effective

# AESCULAP® DS Appendectomy-Clip

Innovative, reliable, convenient,  
cost effective

- **Innovative**

This system is innovative and unique in the segment of Titanium Ligation-Clips

- **Reliable**


The DS-Clip provides a firm hold on the appendix. For reference, please compare the existing Clip Compendium

- **Convenient**

The system is easy to use. It does not require an adaption of the order of operation

- **Cost effective**

A cost-effective system due to reusable clip appliers

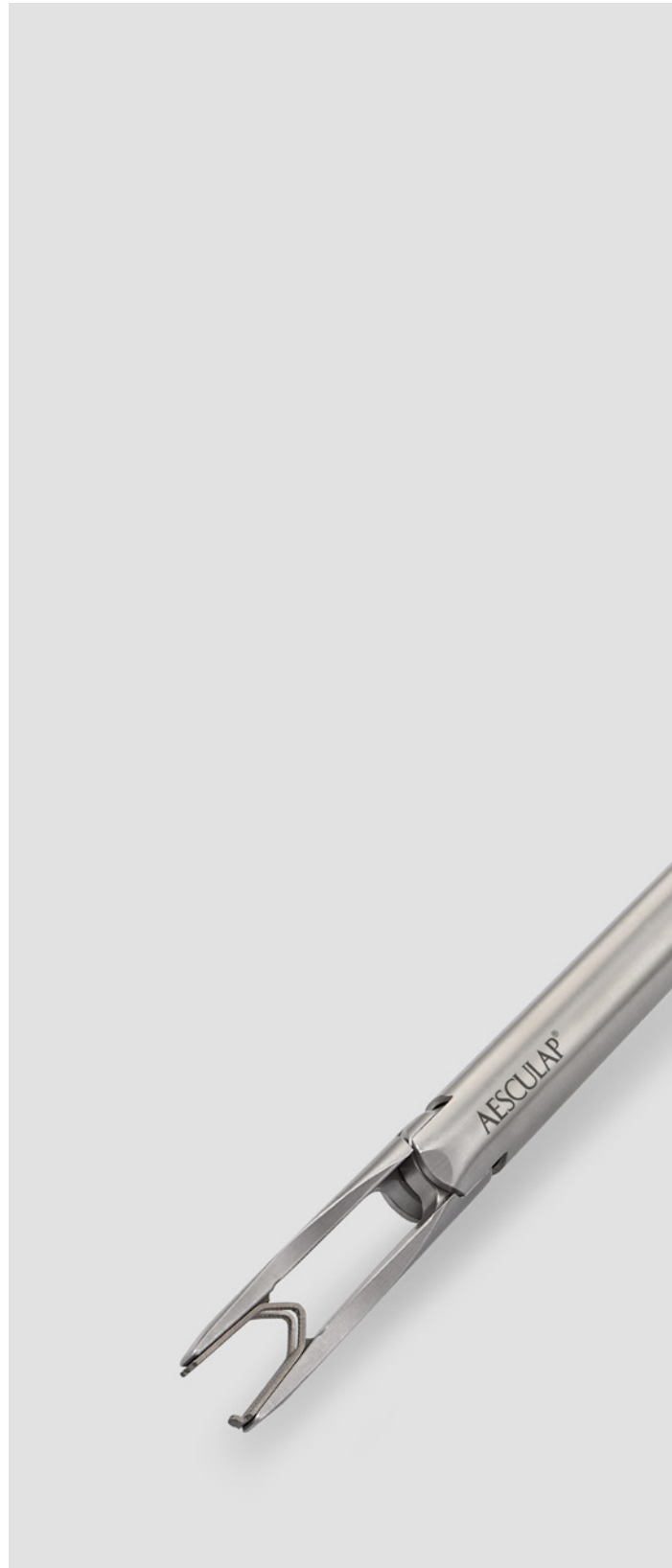



For more information see  
brochure No. D-MIS23002  
Clip Compendium



DS (Double-shank) Appendectomy-Clip

# AESCULAP® DS Appendectomy-Clip



Closing behavior of the clip

## Double-shank clip

- the DS-Clip has a clip shape with parallel arranged bars with a gap between which compresses the tissue. They stabilize the clip against axial dislodgement
- a latch at the distal end of the clip additionally prevents the clip from slipping
- the clip closes from the tip
- a diamond-shaped inner clip profile leads to an enlarged contact surface with the tissue
- the clip is made out of pure titanium



## Clip removal from the cartridge

- simple clip removal
- simplified identification of empty slots by protruding wings
- lateral indentations on the cartridge for a comfortable handling
- adhesive surface at the cartridge back for precise fixation

# AESCULAP® DS Appendectomy-Clip



## Clip applicators

- availability of laparoscopic and open-surgical applicators
- SQ.line® laparoscopic clip applier with integrated overload protection. Constant force distribution on the clipped structure (1)
- 360° rotatable shaft for accurate alignment of the clip
- easy disassembly of the SQ.line® laparoscopic clip applier for cleaning
- well-known, proven and valued Aesculap Quality
- easy-to-clean instruments

## Clip cartridges

- delivery in cartridges à 4 clips and 12 cartridges per sales unit
- the supply is carried out in dispenser boxes, which can be stored in the OR
- sterile supply of the clips

(1) Internal jaw part surface pressure test

| SQ.line® Laparoscopic Clip Applier<br>(complete applier) | Shaft incl. push rod     | Push rod      | Handle        |
|----------------------------------------------------------|--------------------------|---------------|---------------|
|                                                          |                          |               |               |
| length 330 mm                                            | 330 mm                   | 330 mm        |               |
| <b>PQ905R</b><br>Ø 12 mm                                 | <b>PQ915R</b><br>Ø 12 mm | <b>PQ961R</b> | <b>PQ960R</b> |

| Art. No. | DS Appendectomy Clip<br>(titanium ligation clip size X-Large) | Clips per<br>cartridge | Cartridges<br>per pack |  |
|----------|---------------------------------------------------------------|------------------------|------------------------|--|
| PL475SU  | <br>11.5 mm<br>16.2 mm<br>1/1                                 | 4                      | 12                     |  |

| Art. No. | Jaw part angulated | Open-surgical clip appliers |
|----------|--------------------|-----------------------------|
| FC230R   | <br>25°            | <br>length 280 mm           |
| FC231R   | <br>65°            |                             |

# AESCULAP® – a B. Braun brand

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany  
Phone +49 7461 95-0 | [www.bbraun.com](http://www.bbraun.com)

The main product trademark "Aesculap" and the product trademark "SQ.line" are registered trademarks of Aesculap AG. Subject to technical changes. All rights reserved. This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.