

AESCULAP® Container Seals:

Best Practices for Closing and Opening

General information for our valued customers

We are committed to supporting you in your daily operations. To ensure smooth handling and reliable process control, we would like to share some best practices on how to properly close and open the AESCULAP® container seals JG738, JG739, JG752 and JG753.

These best practices apply to the following AESCULAP® container types:

Mini, Dental, Basis standard sizes, PrimeLine® Pro, XXL Endo, Wide-Body, and Scope containers

Closing the container seals



- 1 Close the latch to secure the lid to the container bottom. Insert the tamper-evident seal through the channel. The indicator should be facing up (away from the container).




- 2 Now insert the end (arrow) into the designated slot until it clicks and locks in place.



- 3 Gently pull on the seal to confirm it is fully fastened and secure. Repeat on the other side of the container.

Opening the container seals

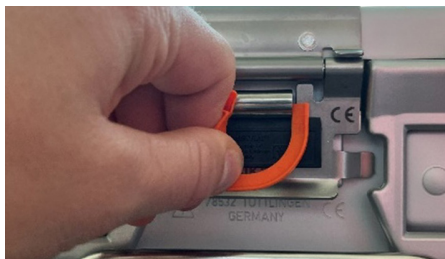

- 1 Break the seal by simply pulling it away with the finger until it releases.



- 2 Before removing the lid, discard all broken pieces of the seals.

If you have any questions or need further assistance, please contact your Aesculap representative.

AESCULAP® – a B. Braun brand

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 7461 95-0 | www.bbraun.com